

# The ABC's of Propaq CS®

<p><b>A</b></p> <p>Age Ranges</p>	<p><b>Neonate:</b> &lt;44 weeks gestational age <b>Pediatric:</b> &gt;44 weeks gestational age to &lt;9 years old <b>Adult:</b> &gt;9 years old</p>
<p><b>B</b></p> <p>Battery Info</p>	<p>Sealed, gel-type lead acid Duration:</p> <ul style="list-style-type: none"> <li>• Monitor only: 2 hours</li> <li>• Monitor+SpO2: 5 hours</li> <li>• Monitor/Printer/SpO2/CO2: 3hours</li> </ul> <p>1hour charging = 2 hours battery life Should be plugged in when not in use. Charges to full capacity in 8-12 hours with instrument on, 6-8 hours if it is off.</p>
<p><b>C</b></p> <p>Cleaning</p>	<p><u>Device</u> Wipe with a nearly-dry cloth moistened with an approved cleaning solution:</p> <ul style="list-style-type: none"> <li>• Warm water, liquid soap, Coverage®, Windex®, Ovation®, Hydrogen peroxide, Wex-cide®, T.B.Q.®</li> </ul> <p>After cleaning, remove excess solution with a soft cloth dampened with clean water, then dry. Do not let solution get into connector openings or crevices. Do not autoclave or immerse.</p> <p><u>Accessories</u> Consult manufacturer's instructions.</p>
<p><b>D</b></p> <p>Defaults</p>	<p>To change from the factory default settings to customized settings, use the following steps:</p> <ol style="list-style-type: none"> <li>1. Be sure you are in the appropriate mode (Adult, Ped, Neo) for which you wish to change the settings.</li> <li>2. Make the desired changes on the device (i.e. Limits, HR/PR tone OFF, etc).</li> <li>3. From the Main Menu/Home Menu, touch SETUP →MORE →CHANGE →SETUP.</li> <li>4. Move cursor to highlight the appropriate mode (Adult, Ped, Neo) under the CUSTOM category.</li> <li>5. Press POWERUP, YES, then SAVE, and YES again.</li> <li>6. Power cycle monitor to ensure that desired changes have taken effect.</li> </ol>
<p><b>E</b></p> <p>End Monitoring</p>	<p><u>Standby/Temporary Disconnect</u></p> <ol style="list-style-type: none"> <li>1. Remove cables from side of monitor.</li> <li>2. Press any of the touch screen keys to acknowledge equipment alert as prompted.</li> <li>3. Acuity waveform tile will turn <b>BLUE</b> to indicate stand-by state.</li> <li>4. Upon return of patient, plug cables back into device. You may be asked to re-confirm patient depending upon length of time patient is off the monitor.</li> </ol> <p><u>Power Down/Permanent Disconnect</u></p> <ol style="list-style-type: none"> <li>1. Touch the <b>NET OFF</b> (touch key, top left corner of screen)</li> <li>2. Turn off monitor (grey recessed button on right side of monitor)</li> </ol> <p>Note: If the device is not turned off after pressing the NET OFF touch key, it will try to reconnect to Acuity.</p>