

Spot Vital Signs® Battery Upgrade Install Instructions

Mat. 720744, 80018655 Ver. C
© 2013 Welch Allyn, Inc.
www.welchallyn.com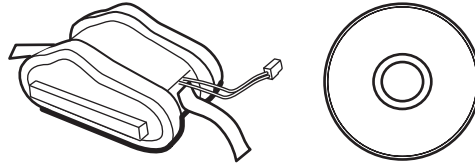


1



①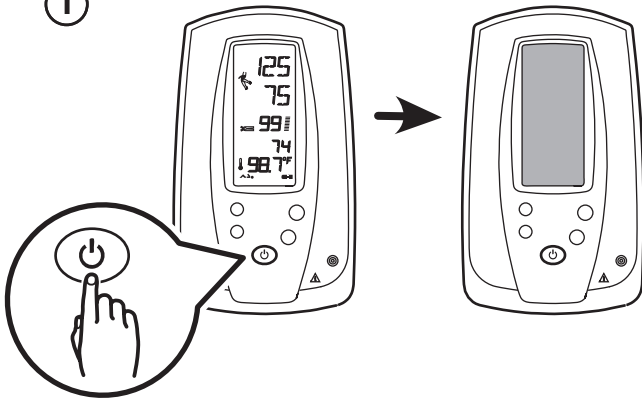


②

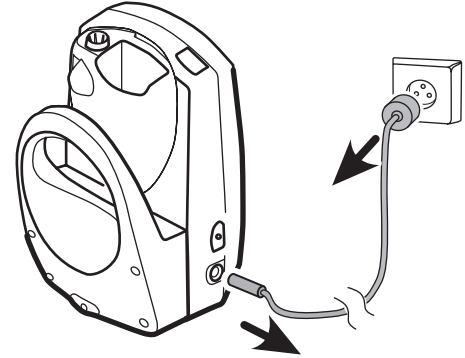
2

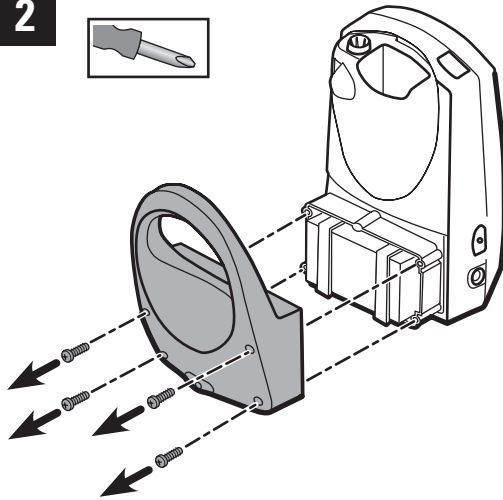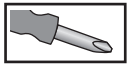
3

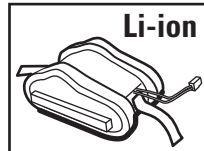
①


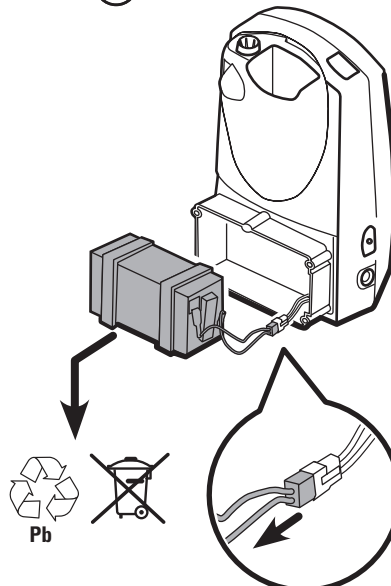
②

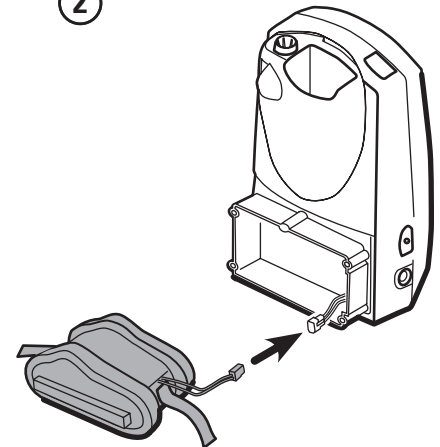
4

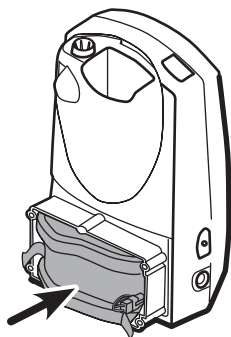
5

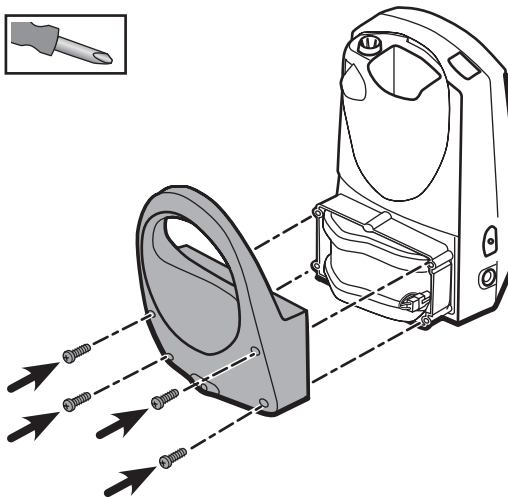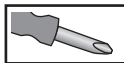6

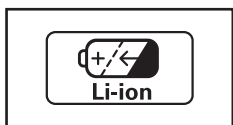
1


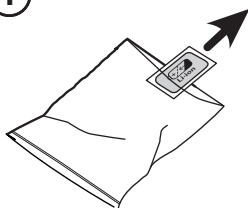
2

3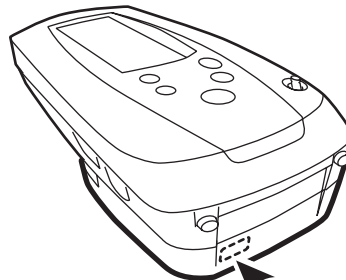

7

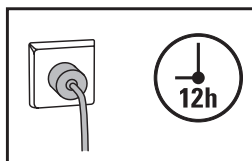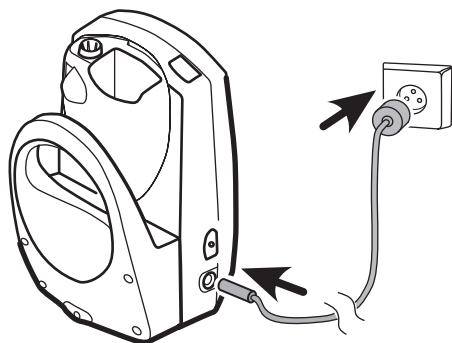English

Set the date and time

After recharging a dead battery or after disconnecting the battery for a few minutes, you must program the date and time screen.


You can change several device operating parameters in the Internal Configuration Mode. When changed, these settings become the default power-up settings. You will also see non-changeable device configurations for technical service purposes.

To enter the Internal Configuration Mode:

1. Turn the Spot Vital Signs off ().
2. Press and hold the **Blood Pressure Start/Stop** () and **Power** () buttons. The device enters the Internal Configuration Mode and displays the software version.


If the device displays E38, press the **Next Patient/Clear/Cancel** () button to cancel the error.

3. Press the **Mode** () button to cycle through the Internal Configuration menu until you see the date option displayed on the screen.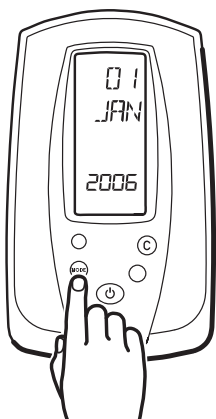

4. Use the **Next Patient/Clear/Cancel** () or **Blood Pressure Start/Stop** () buttons to change the default setting.

5. Press the **Mode** () button once to save the change. Repeat steps 4 and 5 to set the day, month, year, hour, and minute values.

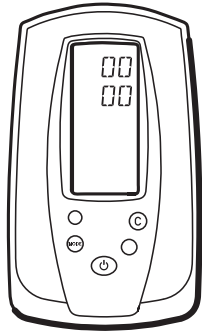6. Press the **Power** () button to exit the Internal Configuration Mode.

Configuration Menu Options

Setting	Description
Blood Pressure Calibration Displays "Cal"	Prepares the Spot Vital Signs for calibration. Only qualified personnel should verify the Spot Vital Signs blood pressure calibration.
Inflation Pressure Preset Level Displays "PrP"	120, 140, 160, 180, 200, 240, 280 mmHg. Factory default is 160 mmHg.
Pressure Preset Level Displays "PrP"	On or off. Disables or enables the front panel Pressure Preset button.
Backlight Displays "BLT"	On or off.
Mean Arterial Pressure Displays "MAP"	On or off.
Date/Time	Changes or updates the actual date and time.
Temperature Scale Displays "TMP MOD"	Fahrenheit (°F) or Celsius (°C) Normal Mode / Fahrenheit (°F) or Celsius (°C) Monitor Mode
Blood Pressure Units Displays "BP"	mmHg or kPa.
Battery Readings Displays "BAT"	Displays the total battery voltage.
Battery Life Displays "LFE"	Total number unit measurements. Displayed information only; operator cannot change.

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