Mat. 722977, Ver. A

Dear Welch Allyn CP 50™ or CP 50 Plus™ 12-lead electrocardiograph owner:


Enclosed is a new software update, version 01.30.04, to improve the functionality of your electrocardiograph.

Improvements include:

- Colored grid for PDF output
- Brazilian Portuguese language support
- Physician and comments entry support
- Chinese age unit of measurement correction (for China only)
- Additional global measurements (for China only)
- First and last name support for the Basic model

# Overview

This document describes the steps required to correctly update CP 50 devices to the latest software version using the UpdateUtility. It is important that you follow the following flow chart to identify the correct section to use when performing your update. The flow chart will determine the correct software update kit to use.

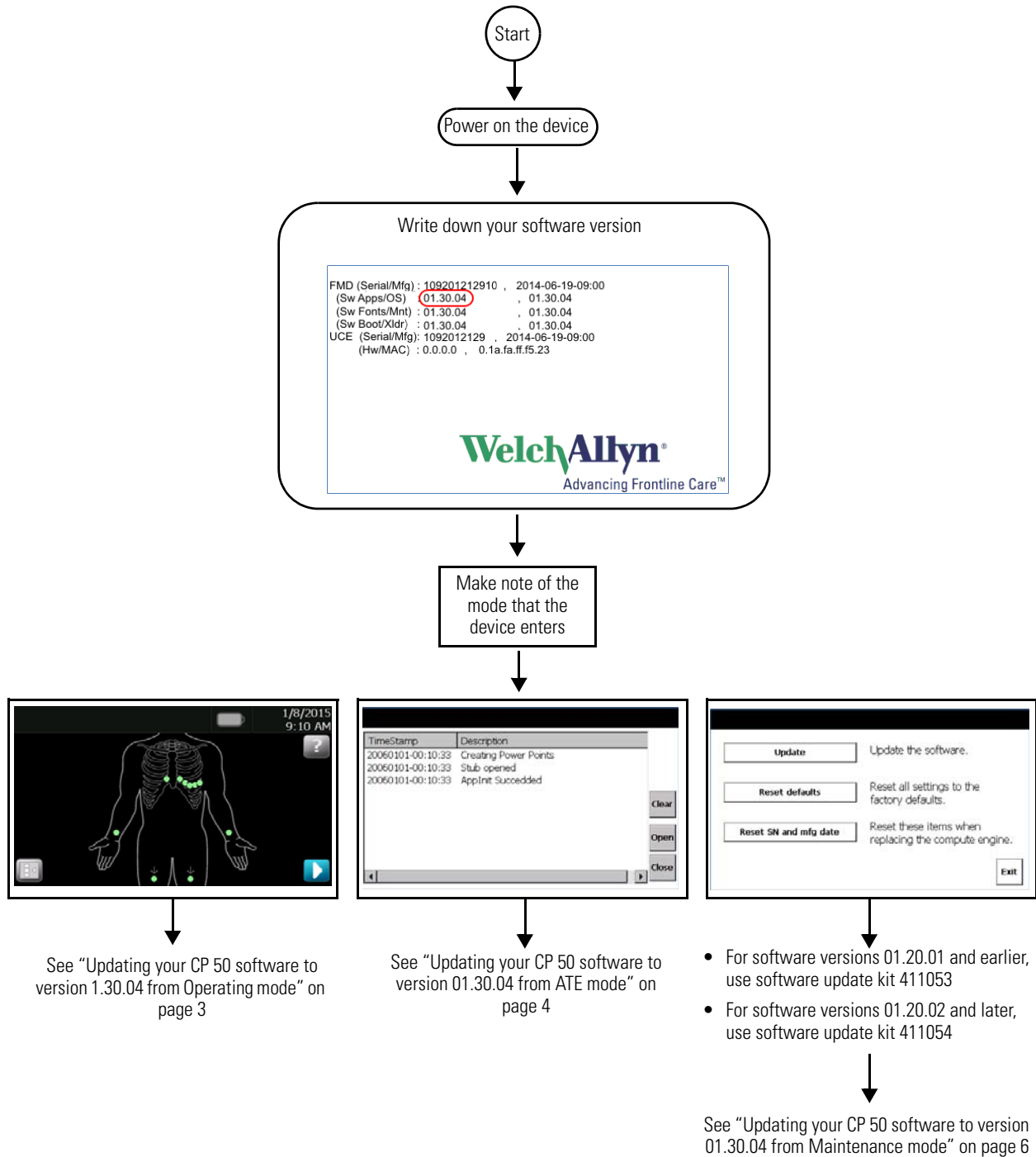# Updating your CP 50 software to version 1.30.04 from Operating mode

**Caution** Data loss risk. Do not power down or interrupt the update process.

## Before you begin

To perform the update, you will need the following:

- The USB drive (part number 411053) that contains the update utility.
- The CP 50 you plan to update.
- A cable to connect the CP 50 to AC power.
- An AC power outlet.

## Update the software

1. Make sure the system is powered down. To power down the CP 50, press and hold the power button for 6 seconds.
2. Connect the CP 50 to AC power.
3. Plug the USB drive (part number 411053) into the CP 50.
4. Power up the CP 50 by pressing the power button. The software update begins after the device powers up.

**Note** The device will restart multiple times if the current software version 01.20.03 or later. The software update will operate on all earlier versions.

5. When the software update completes, the device might prompt you for the serial number and the manufacturing date. Enter these values and click **Update**.

**Note** The serial number is located on the label in the rear of the device. The current date can be entered if the manufacturing date is unavailable.

6. Remove the USB drive after the update is complete. Press **OK** to restart.

**Caution** Do NOT remove AC power from the device during the restart.

7. Once the device has completed the restart process, select your language.

## Verify the software version:

- a. Go to **Menu > Settings (System tab) > About**.
- b. Verify that the correct software version is listed for the CP 50.


# Updating your CP 50 software to version 01.30.04 from ATE mode

**Caution** Data loss risk. Do not power down or interrupt the update process.

## Before you begin

In ATE mode, the following screen is displayed when the device starts up.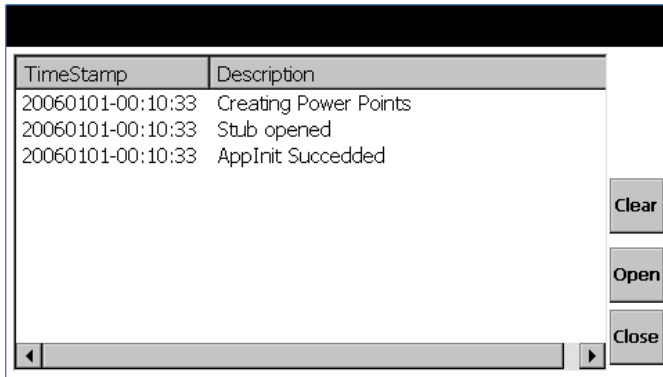

To perform the update, you will need the following:

- The USB drive (part number 411053) that contains the update utility.
- The CP 50 you plan to update.
- A cable to connect the CP 50 to AC power.
- An AC power outlet.

## Update the software

1. Make sure the system is powered down. To power down the CP 50, press and hold the power button for 6 seconds.
2. Connect the CP 50 to AC power.
3. Plug the USB drive (part number 411053) into the CP 50.
4. Power up the CP 50 by pressing the power button. The software update begins after the device powers up.

**Note** The device will restart multiple times if the current software version 01.20.03 or later. The software update will operate on all earlier versions.

5. When the software update completes, the device might prompt you for the serial number and the manufacturing date. Enter these values and click **Update**.

**Note** The serial number is located on the label in the rear of the device. The current date can be entered if the manufacturing date is unavailable.

6. Remove the USB drive after the update is complete. Press **OK** to restart.

**Caution** Do NOT remove AC power from the device during the restart.

7. Once the device has completed the restart process, select your language.

### Verify the software version:

- a. Go to **Menu > Settings (System tab) > About**.
- b. Verify that the correct software version is listed for the CP 50.


# Updating your CP 50 software to version 01.30.04 from Maintenance mode

**Caution** Data loss risk. Do not power down or interrupt the update process.

## Before you begin

In Maintenance mode, the following screen is displayed when the device starts up.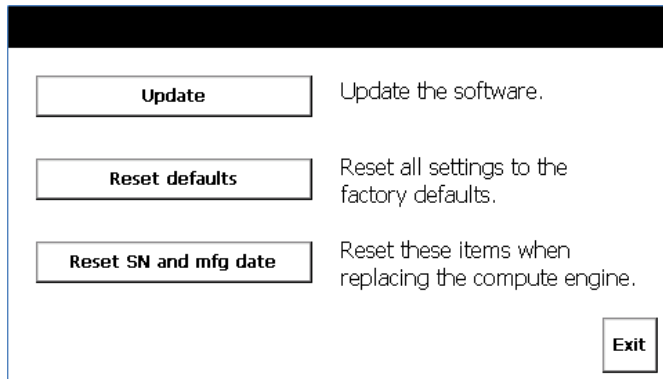

**Note** Ensure that you make note of the device software version displayed on the splash screen.

**Note** If the current version of the device is 01.20.01 or earlier, use the USB drive with the part number of 411053. If the current version of the device is 01.20.02 or 01.20.03, use the USB drive with the part number 411054.

To perform the update, you will need the following:

- The USB drive that contains the update utility.
- The CP 50 you plan to update.
- A cable to connect the CP 50 to AC power.
- An AC power outlet.

## Update the software

1. Make sure the system is powered down. To power down the CP 50, press and hold the power button for 6 seconds.
2. Connect the CP 50 to AC power.
3. Plug the USB drive into the CP 50.
4. Power up the CP 50 by pressing the power button.
5. Press **Update**.
6. Select the Xloader and press **Next**.
7. The screen displays the current and new versions. Press **Update**.
8. After update completes, press **OK** followed by the **Back** button. Repeat step 5 and step 6 for the remaining binary files:
  - Bootloader
  - Maintenance Image
  - Main OS
  - Main Fonts
  - Main Applications
9. After loading all the binary images, press **Next**.
10. The system is at the Maintenance screen. Press **Exit** and the system will restart within two seconds.

**Note** The device will restart multiple times if the current software version 01.20.03 or later. The software update will operate on all earlier versions.

11. When the software update completes, the device might prompt you for the serial number and the manufacturing date. Enter these values and click **Update**.

**Note** The serial number is located on the label in the rear of the device. The current date can be entered if the manufacturing date is unavailable.

12. Remove the USB drive after the update is complete. Press **OK** to restart.

**Caution** Do NOT remove AC power from the device during the restart.

13. Once the device has completed the restart process, select your language.

## Verify the software version:

- a. Go to **Menu > Settings (System tab) > About**.
- b. Verify that the correct software version is listed for the CP 50.

Welch Allyn Limited  
Navan Business Park  
Dublin Road  
Navan, County Meath  
República da Irlanda  
[www.welchallyn.com](http://www.welchallyn.com)

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